




MEDEFIL, INC.

DIRECTIONS FOR USE:

Refer figure 1 – figure 9 for pictorial presentation


Solution and fluid path are sterile and non-pyrogenic if the tip cap is in place, syringe is intact and there is no evidence of leakage. Use proper aseptic technique.

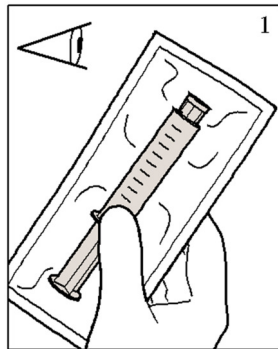


Inspect plastic wrapping. Do not use if plastic wrapping is damaged or not intact




Remove plastic packaging by tearing along perforation

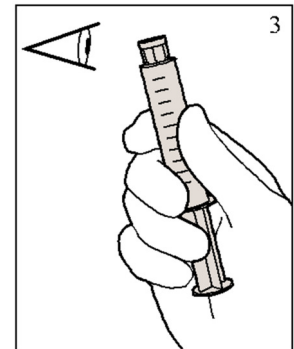


Do not use if, solution is discolored, cloudy, hazy, or contains a precipitate, or if the syringe is damaged

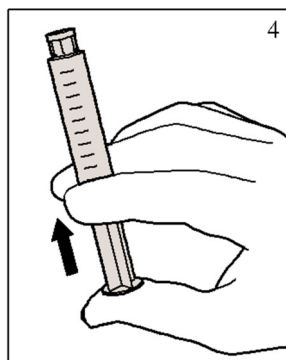


With the tip cap of the syringe on, press the syringe forward to properly activate the syringe. Improper activation may difficult syringe use and/or may cause plunger rod separation. Never draw back rod because the product may become contaminated.

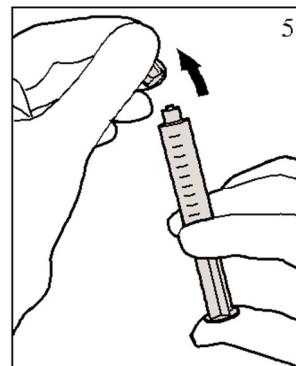


Remove tip cap. Hold the syringe unit upright and prime to expel any air bubble if present

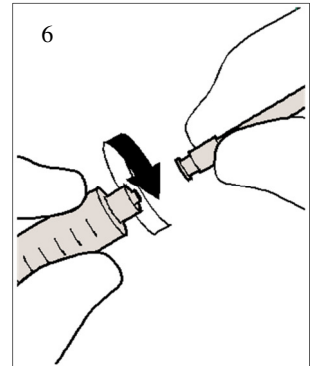


Syringe is now ready to use. Per Institution protocol attach flush syringe to access device and flush

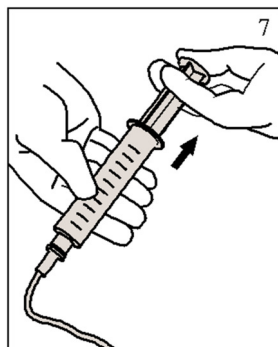

When attempting to aspirate tubing or indwelling device by pulling back on the syringe plunger, while attached to the tubing or indwelling device, use two-handed technique with one hand on the syringe barrel and the other on the plunger. Pull the plunger straight back. Do not pull or bend the plunger sideways

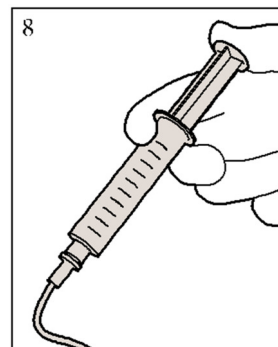

Use in accordance with intravenous tubing or indwelling device manufacturer's recommendation

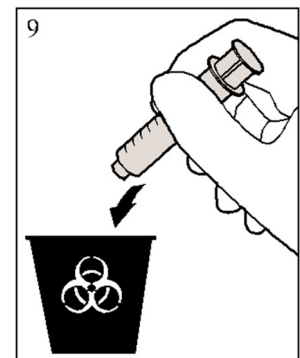

Discard empty unit after use. Discard any unused portion. Do not reuse disposable syringes